

EyeNOVATION®

INNOVATION IM AUGE

Ringject


Preloaded Capsular Tension Ring
in a single use injector




Clockwise implantation



Counter clockwise implantation

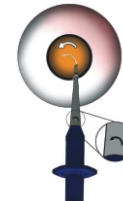

1. Remove the RingJect from blister pack

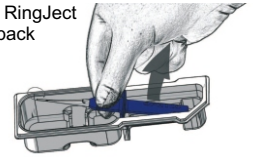

2. Depress the plunger to unlock, stop when you hear the "click"



3. Slowly release the plunger, the Capsular Tension Ring will load in the injector



4. The injector is now loaded and ready for insertion into the eye



Preloaded: Self-loading, single packaging a valuable addition to your surgical armamentarium

Injector: Easy to use one handed technique

Total control Includes action/retraction mechanism

Implant clockwise or counter clockwise Indicators on injector

Long small tip Especially convenient in cases of deep set eyes

Capsular - Circular expansion and stabilization of the capsular bag

Tension - Safe IOL centration in eyes with zonular dehiscence

Ring: Prevents IOL decentration after capsular shrinking

Stabilized conditions during Phaco emulsification surgery

Reduced risk of capsular fibrosis

Improves visual acuity when implanted along with premium IOL

Ringject PC 375 CTR Model 275 12/10

Ringject PC 376 CTR Model 276 13/11

Compression: From 12mm to 10mm

From 13mm to 11mm

Material: PMMA/Flexible Ring

PMMA/Flexible Ring

Available in: Clear PMMA

Clear PMMA

Overall Ø: 12mm

13mm

Heidsieker Heide 90

33739 Bielefeld

tel: 05206 - 6031 - 0